



Instagram




YouTube



Facebook

DISTRIBUTED BY

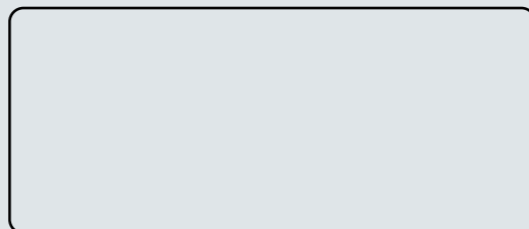

Changzhou Sifary Medcial Technology Co., Ltd
Add: NO.99, Qingyang Road, Xuejia County, Xinbei
District, Changzhou City, 213000 Jiangsu, China
Email: ivy@sifary.com
Web: www.eighteeth.com



TIP BOOK



CATALOG

Scaling

G1



Order Code: 6680523

G1-S



Order Code: 6680617

G2



Order Code: 6680524

G2-S



Order Code: 6680618

G4



Order Code: 6680526

G4-S



Order Code: 6680619

G5



Order Code: 6680527

G5-S



Order Code: 6680620

1

2

3

3

4

Periodontic

P1

Order Code: 6680528



5

P1-S

Order Code: 6680621



P3

Order Code: 6680529



6

P3-S

Order Code: 6680622



P4

Order Code: 6680530



7

P4-S

Order Code: 6680623



7

P52

Order Code: 6680533



8

P52-S

Order Code: 6680626



P50L

Order Code: 6680531



9

P50L-S

Order Code: 6680624



P50R

Order Code: 6680532



10

P50R-S

Order Code: 6680625



P53L

Order Code: 6680534



11

P53L-S

Order Code: 6680627



P53R

Order Code: 6680535



12

P53R-S

Order Code: 6680628



12

P59

Order Code: 6680536



13

P59-S

Order Code: 6680630



P56

Order Code: 6680544



14

P56-S

Order Code: 6680629



P3D

Order Code: 6680654



15

P3-SD

Order Code: 6680636



P4D

Order Code: 6680631



16

P4-SD

Order Code: 6680637



Endodontic

E1

Order Code: 6680537



17

E1-S

Order Code: 6680656



E3

Order Code: 6680538



18

E3-S

Order Code: 6680657



E4

Order Code: 6680539



19

E4-S

Order Code: 6680658



19

E5

Order Code: 6680540



20

E5-S

Order Code: 6680659



E7

Order Code: 6680542



21

E7-S

Order Code: 6680660



E8

Order Code: 6680608



22

E8-S

Order Code: 6680614



E9

Order Code: 6680609



23

E9-S

Order Code: 6680615



E10

Order Code: 6680610



24

E10-S

Order Code: 6680616



24

E3D

Order Code: 6680651



25

E3-SD

Order Code: 6680638



E30

Order Code: 6680611



26

E30-S

Order Code: 6680661



E31

Order Code: 6680612



27

E31-S

Order Code: 6680662



E4D

Order Code: 6680652



28

E4-SD

Order Code: 6680639



E5D

Order Code: 6680653



29

E5-SD

Order Code: 6680640



29

E6D

Order Code: 6680632



30

E6-SD

Order Code: 6680641



E7D

Order Code: 6680633



31

E7-SD

Order Code: 6680642



E8D

Order Code: 6680634



32

E8-SD

Order Code: 6680643



TC1D

Order Code: 6680648



33

TC1-SD

Order Code: 6680650










SCALING

G1
G1-S

Order Code **6680523**
Order Code **6680617**






Function Description: All-round removal of supragingival calculus

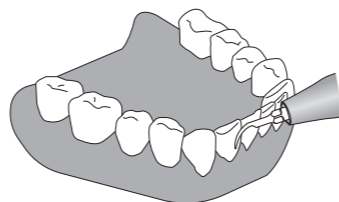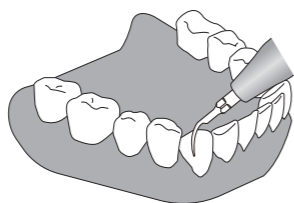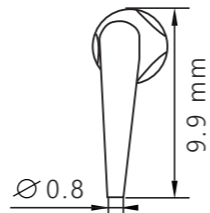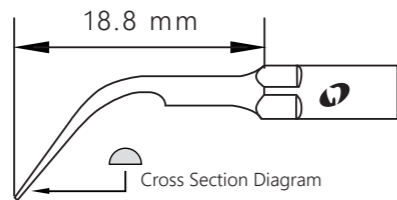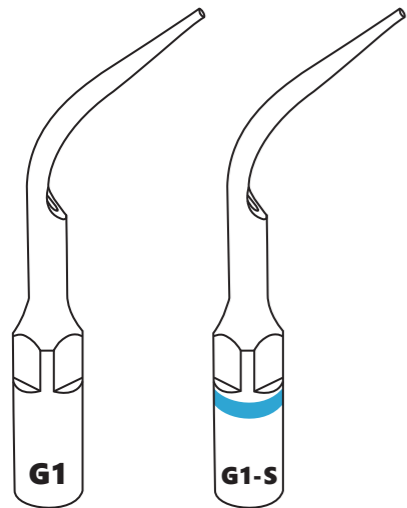

G2
G2-S

Order Code **6680524**
Order Code **6680618**






Function Description: Removal of significant supragingival calculus

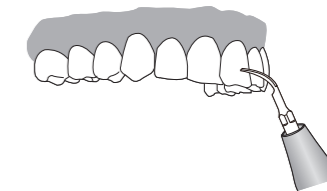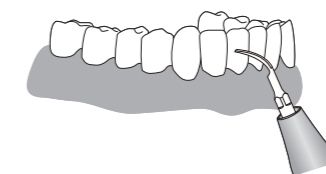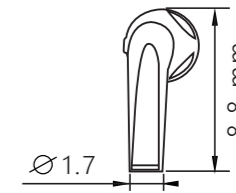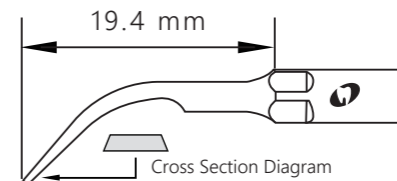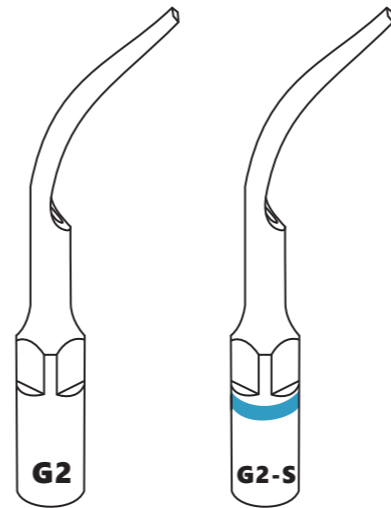

G4
G4-S

Order Code **6680526**
Order Code **6680619**






Function Description: Used for supragingival and interdental space scaling

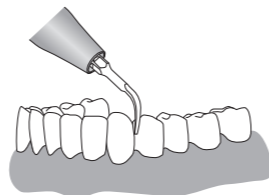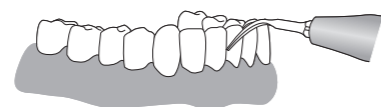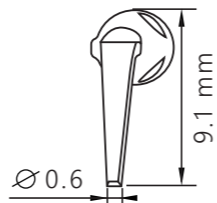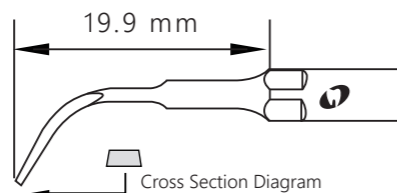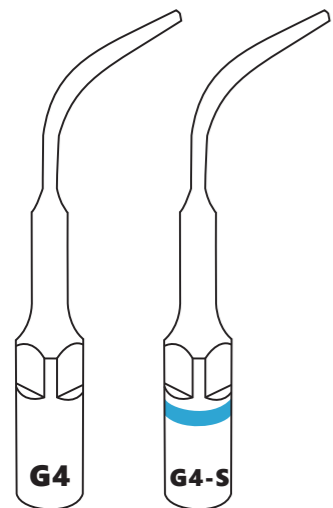

G5
G5-S

Order Code **6680527**
Order Code **6680620**

Function Description: Removal of significant supragingival calculus

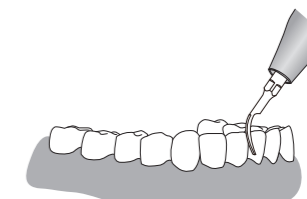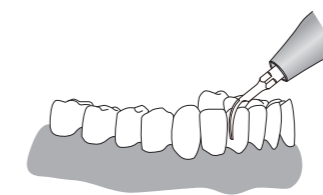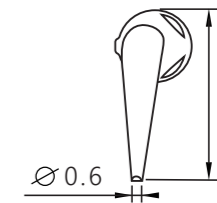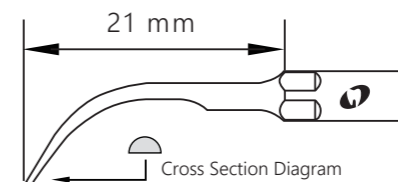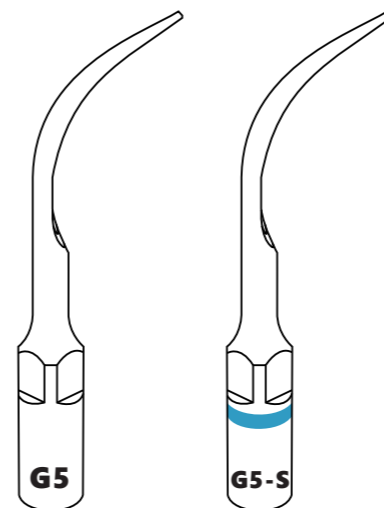

PERIODONTIC








P1
P1-S

Order Code **6680528**

Order Code **6680621**

Function Description: Removal of subgingival calculus and periodontal irrigation

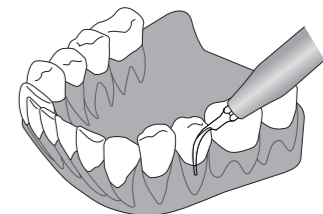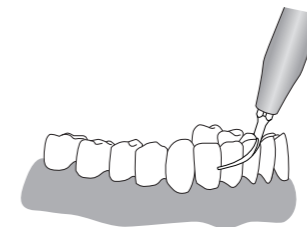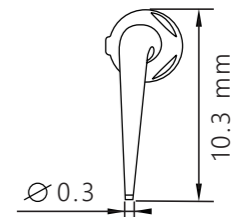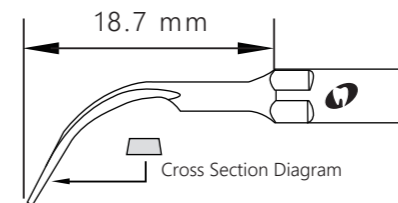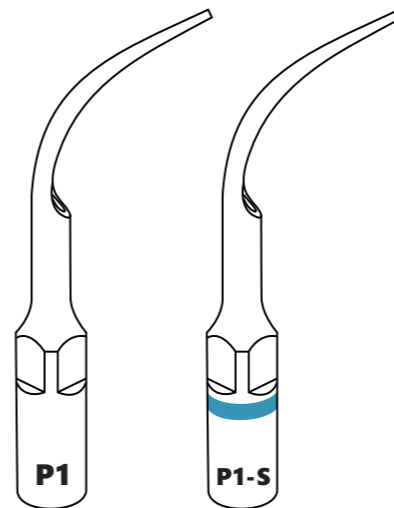

Mode: P
Power: 1-10



Recommended
Service Life
150 Teeth




Water Flow Rate
mid&high

P3
P3-S

Order Code **6680529**
Order Code **6680622**

Function Description: Removal of calculus deep in periodontal pocket and its irrigation

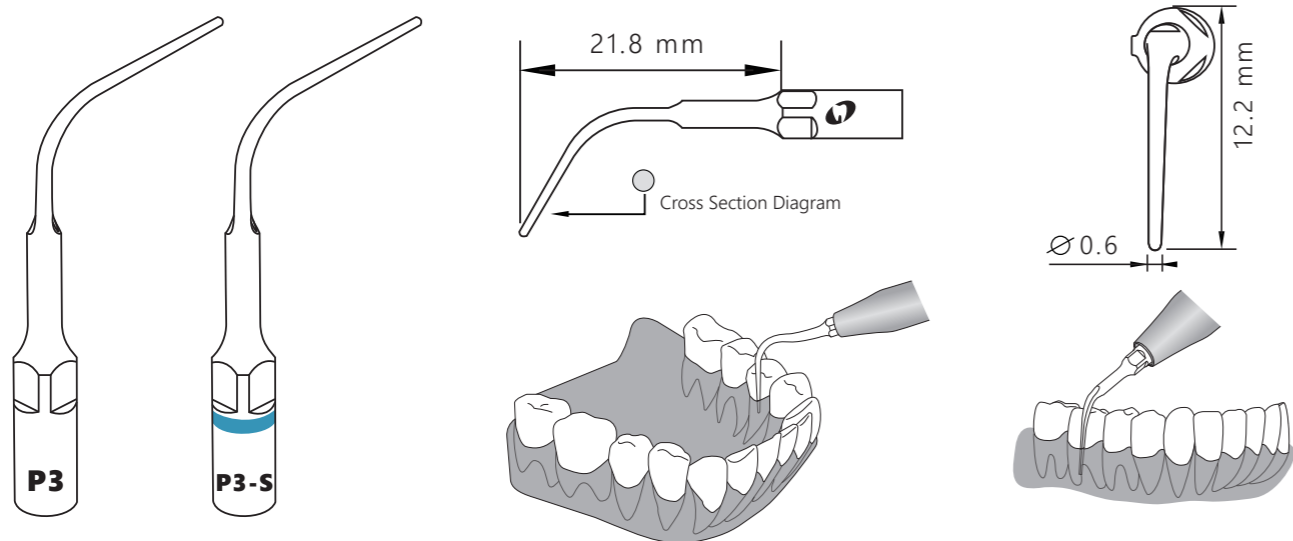

P
Mode: P
Power: 1-6


Recommended
Service Life
150 Teeth

Water Flow Rate
mid&high

P4
P4-S

Order Code **6680530**
Order Code **6680623**

Function Description: Removal of calculus in shallow pockets

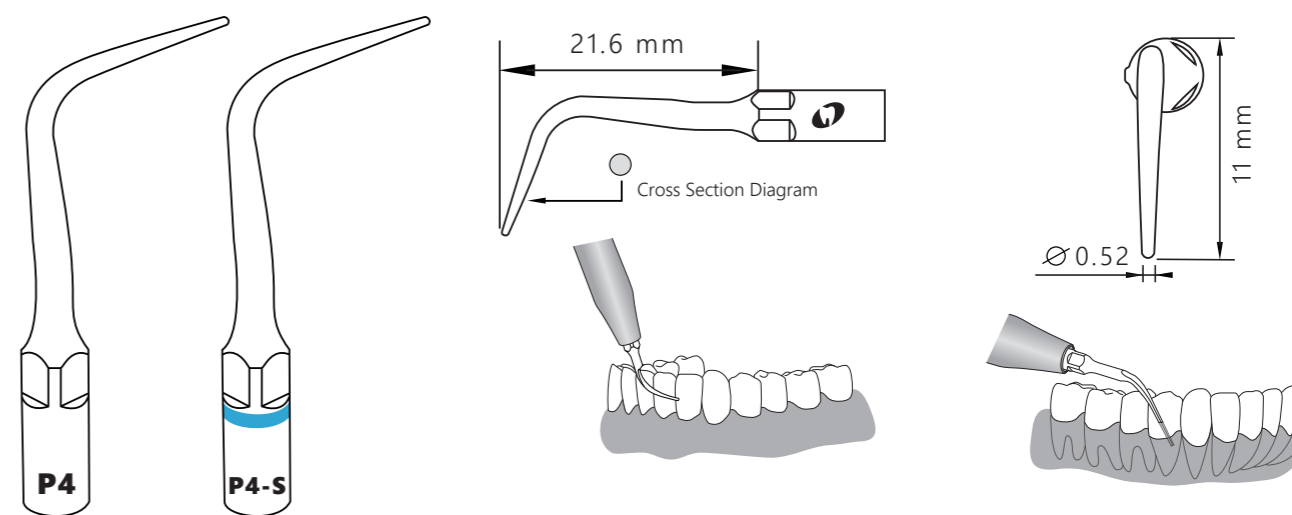

P
Mode: P
Power: 1-6


Recommended
Service Life
200 Teeth

Water Flow Rate
mid&high

P52
P52-S


Order Code **6680533**
Order Code **6680626**

Function Description: Supragingival scaling

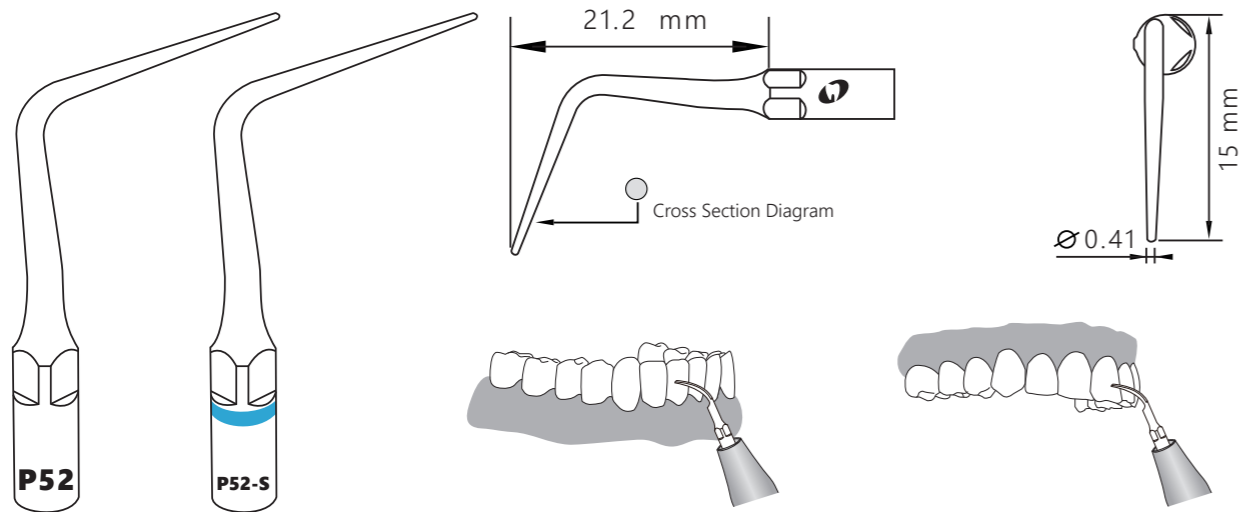

 Mode: P
Power: 1-10


 Recommended
Service Life
80 Teeth

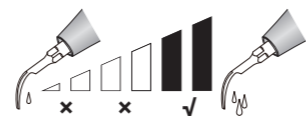 Water Flow Rate
high

P50L
P50L-S

Order Code **6680531**
Order Code **6680624**

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

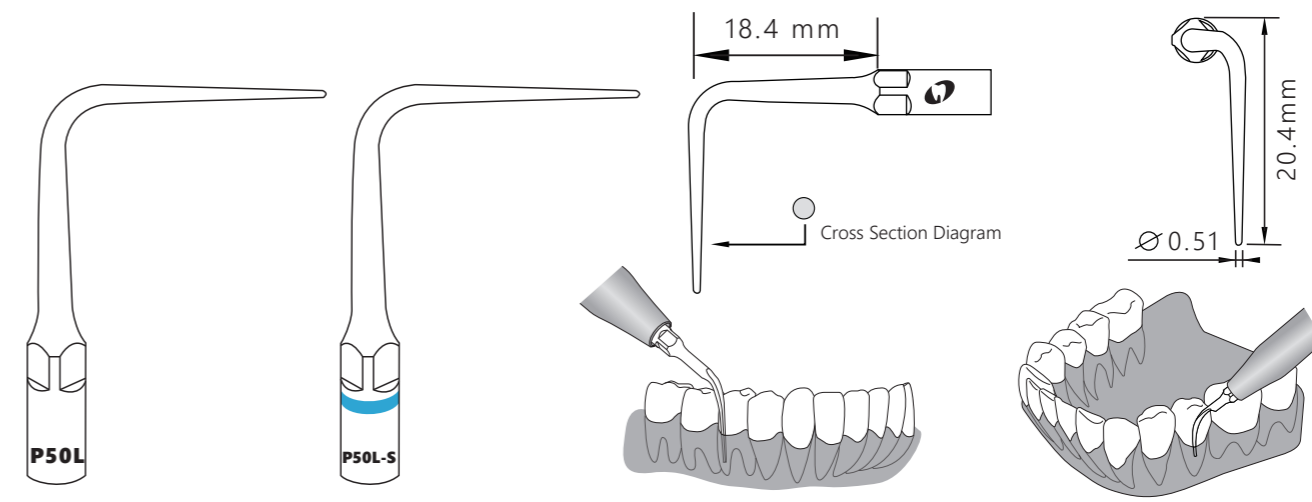

 Mode: P
Power: 1-10


 Recommended
Service Life
80 Teeth

 Water Flow Rate
high

P50R
P50R-S

Order Code **6680532**
Order Code **6680625**

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

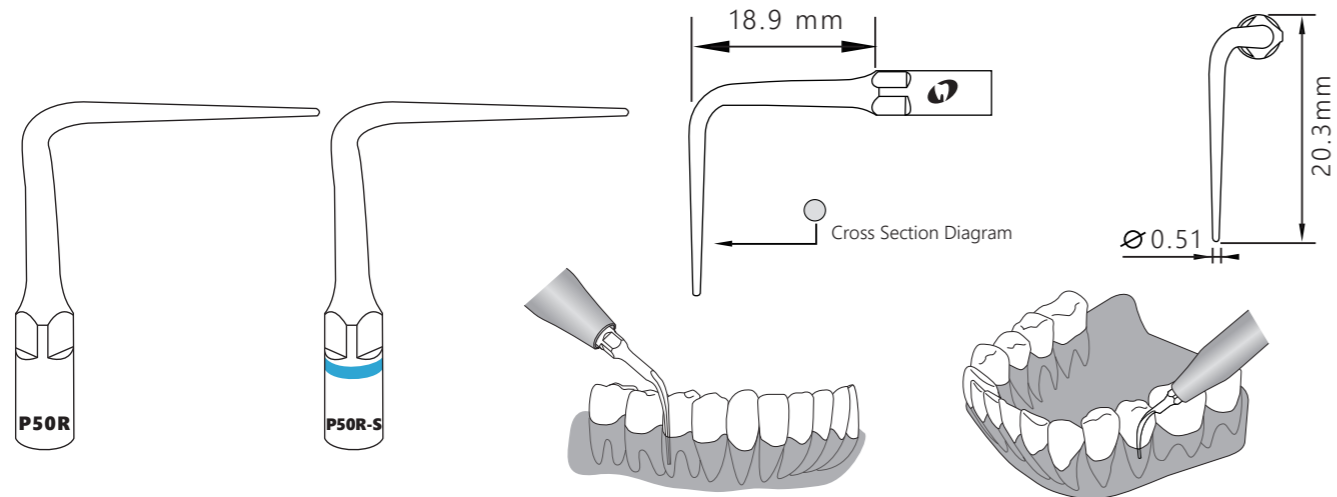

P Mode: P
Power: 1-10


Recommended
Service Life
80 Teeth

Water Flow Rate
high

P53L
P53L-S

Order Code **6680534**
Order Code **6680627**

Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

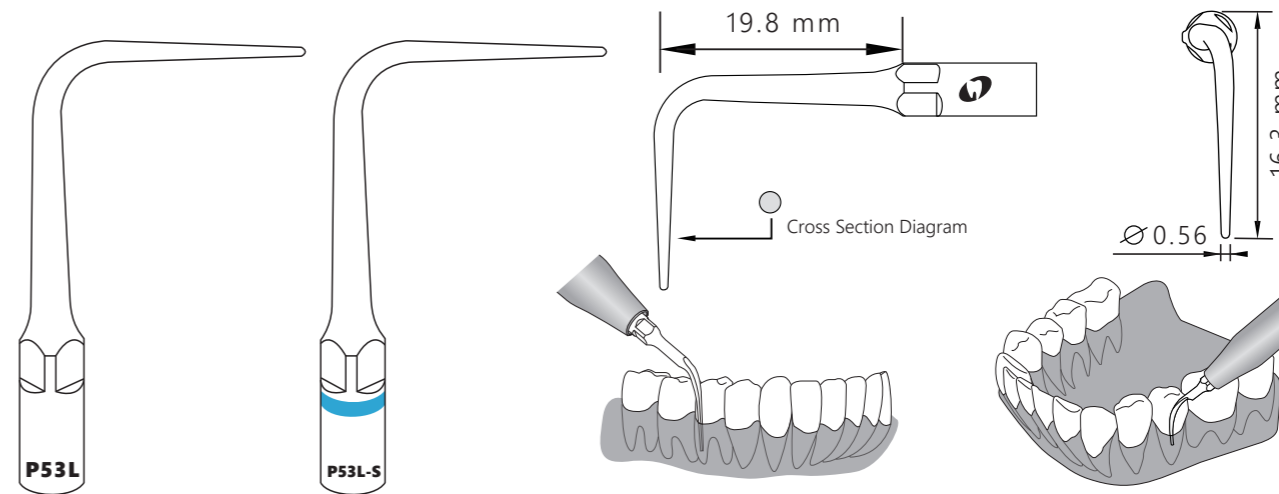

P Mode: P
Power: 1-10



Recommended
Service Life
80 Teeth

Water Flow Rate
mid&high

P53R
P53R-S

Order Code **6680535**
Order Code **6680628**


Function Description: Used for subgingival calculus probing, subgingival proximal surface scaling and root planing

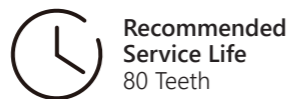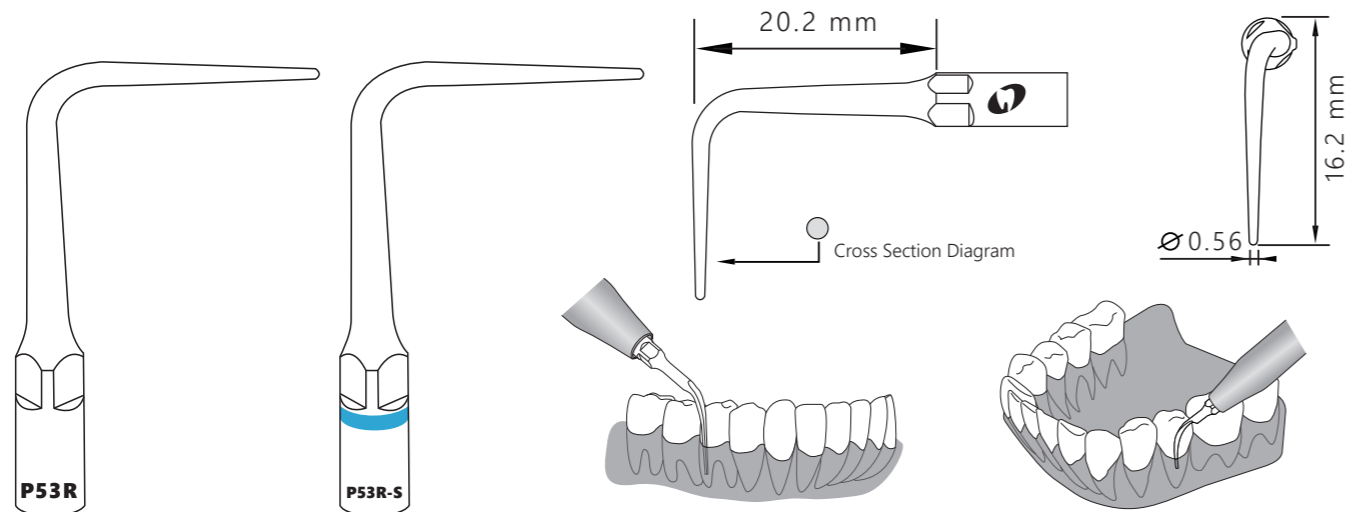

P59
P59-S

Order Code **6680536**
Order Code **6680630**


Function Description: Used for subgingival calculus probing and scaling

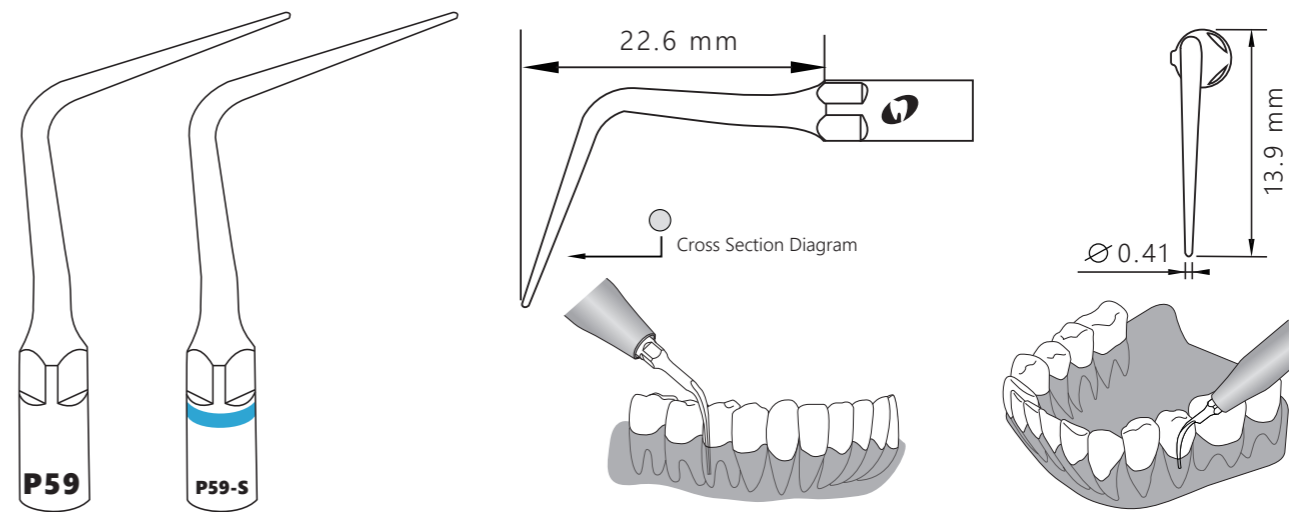

P56
P56-S

Order Code **6680544**
Order Code **6680629**

Function Description: Removal of significant supragingival calculus and plaque

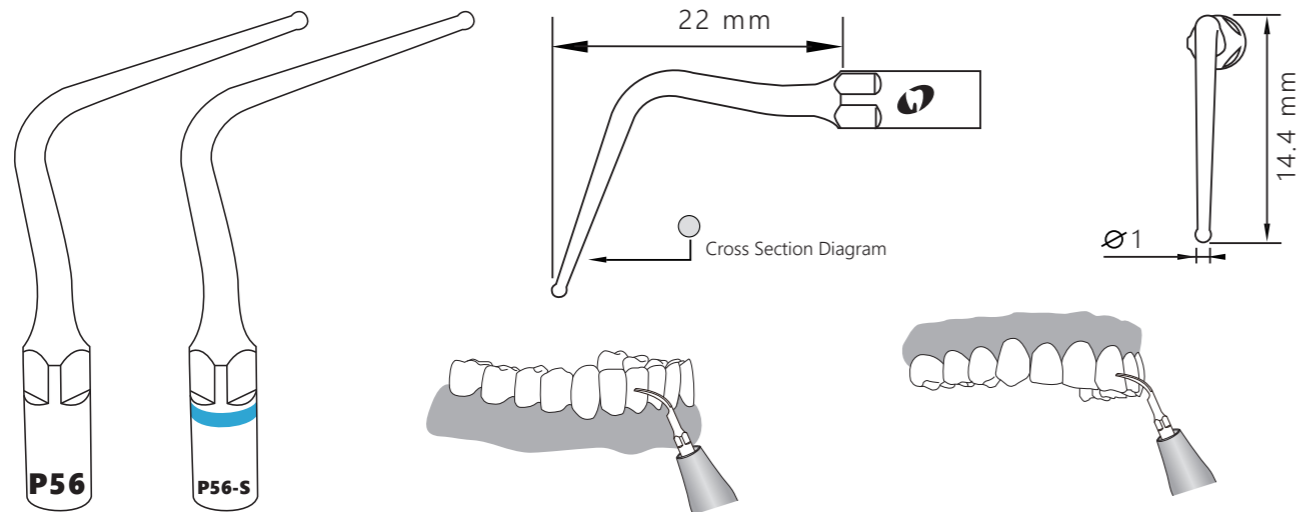

 Mode: P
Power: 1-10


 Recommended
Service Life
80 Teeth

 Water Flow Rate
high

P3D
P3-SD

Order Code **6680654**
Order Code **6680636**

Function Description: Used to level off the surface of endo after the periodontal flap surgery

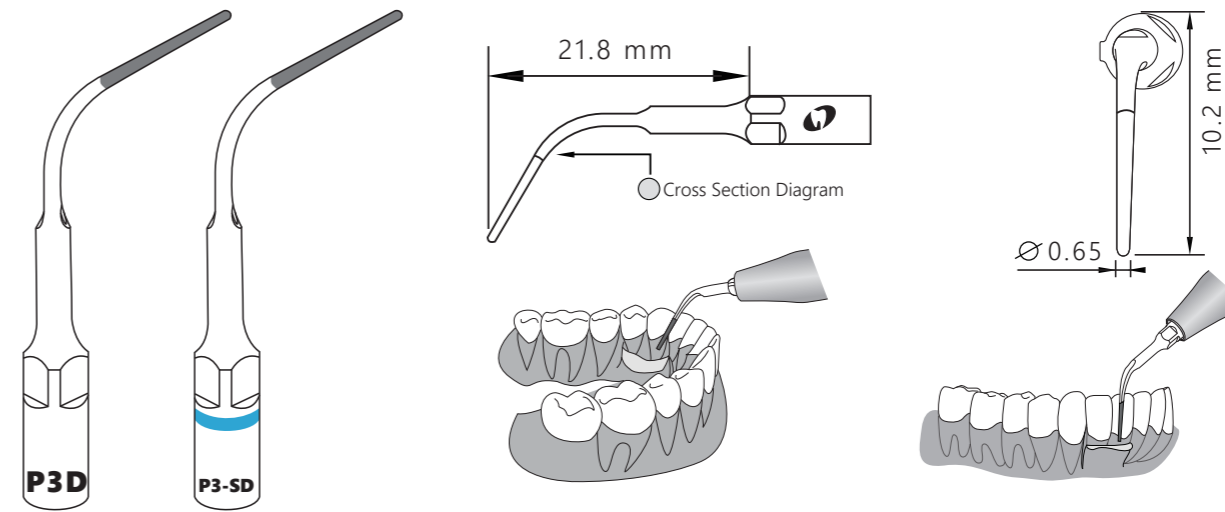

 Mode: P
Power: 1-6



 Recommended
Service Life
50 Teeth

 Water Flow Rate
mid&high

P4D
P4-SD

Order Code **6680631**
Order Code **6680637**

Function Description: For locating root canal and removal of calcification on the one-third of crown

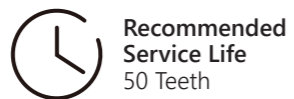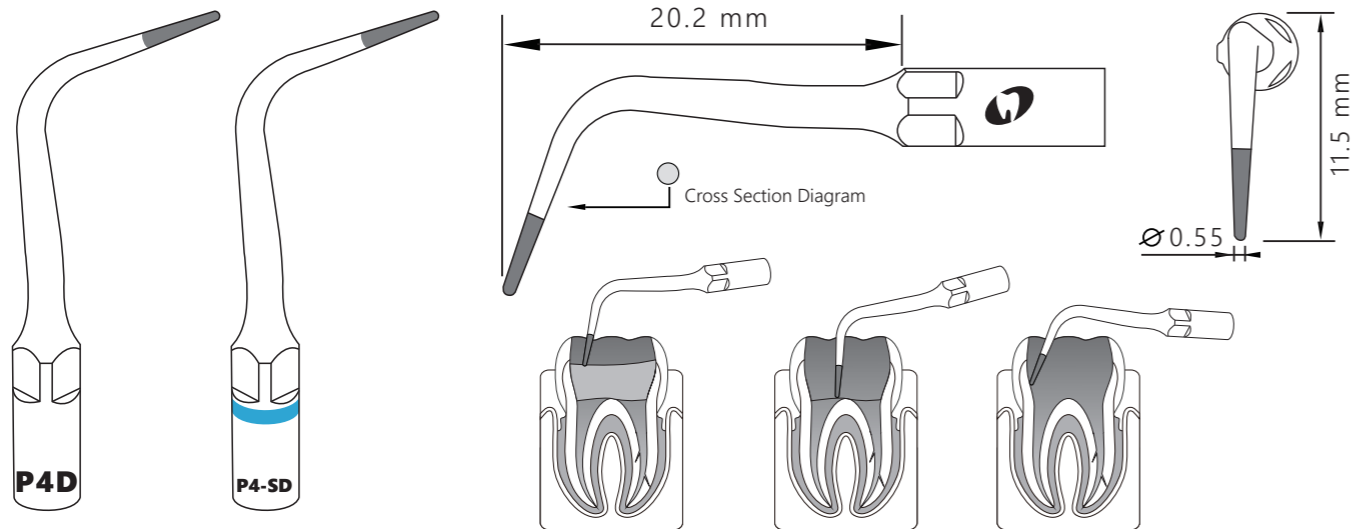


ENDODONTIC

E1
E1-S

Order Code **6680537**
Order Code **6680656**

Function Description: 120° angle holder for 0.7mm-0.8mm endo file for root canal cleaning

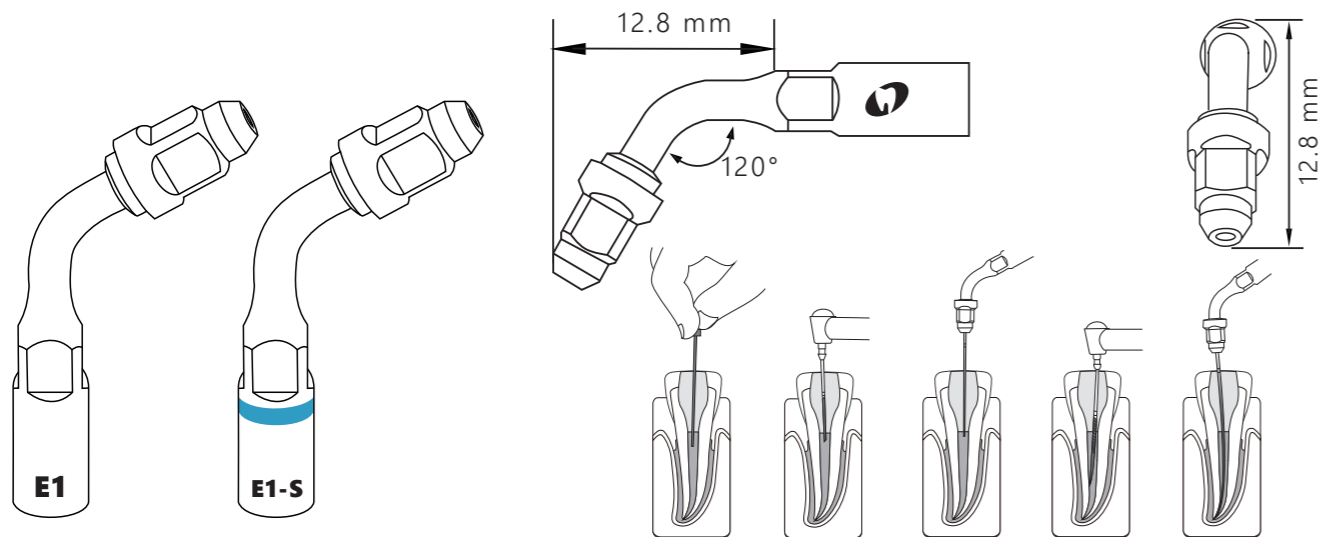

E Mode: E
Power: 1-3


Recommended Service Life
500 times

Water Flow Rate
mid&high

E3
E3-S

Order Code **6680538**
Order Code **6680657**

Function Description: For lateral condensation of gutta percha by heating effect

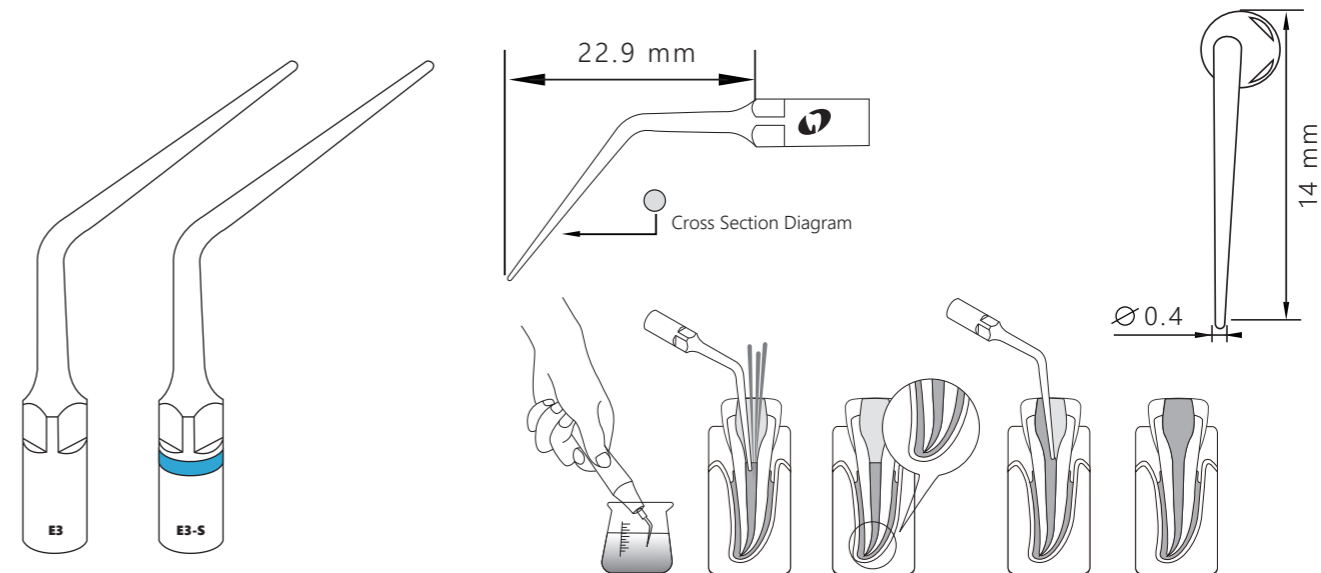

E Mode: E
Power: 1-3


Recommended Service Life
500 root canals

Water Flow Rate
No water

E4
E4-S


Order Code **6680539**
Order Code **6680658**

Function Description: Removal of foreign filling material during re-treatment

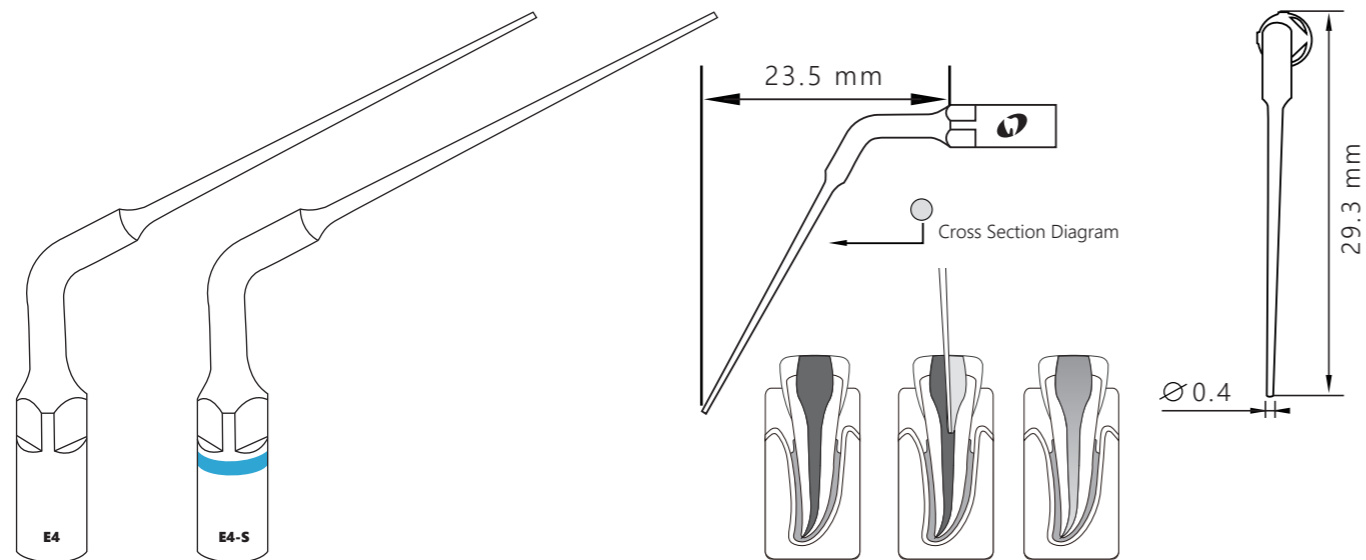

 Mode: E
Power: 1-6


 Recommended
Service Life
100 root canals

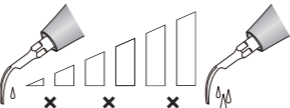 Water Flow Rate
No water

E5
E5-S


Order Code **6680540**
Order Code **6680659**

Function Description: Removal of foreign filling material during re-treatment

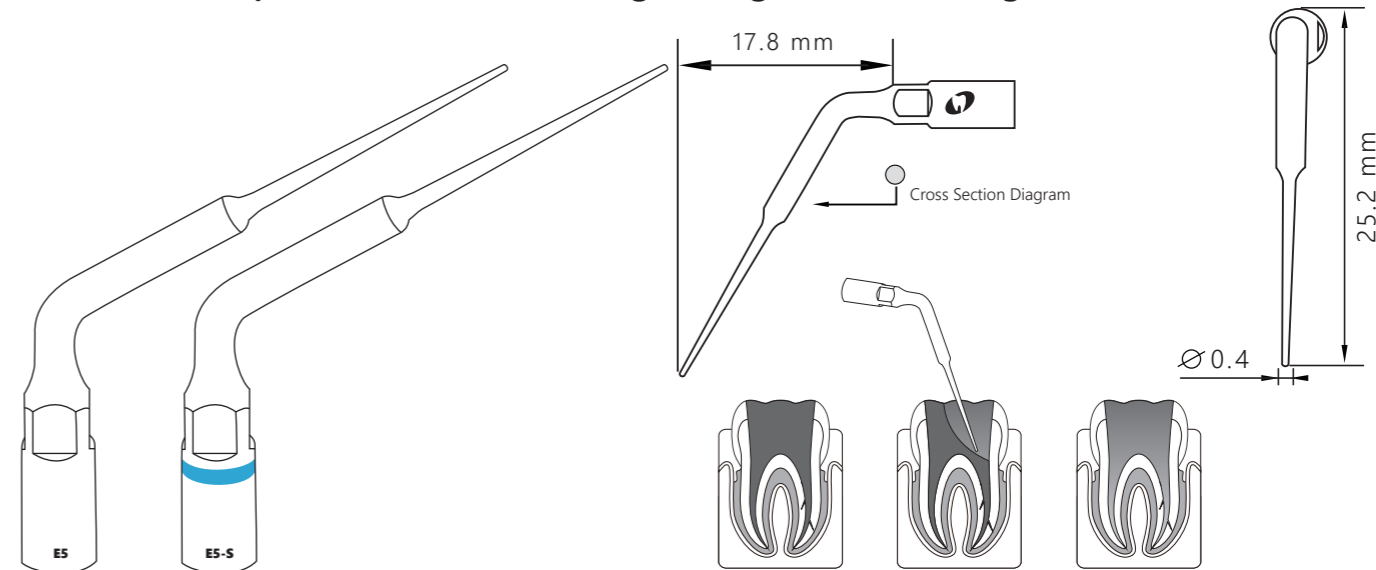

 Mode: E
Power: 1-6


 Recommended
Service Life
100 root canals

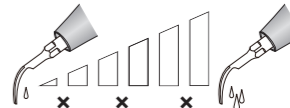 Water Flow Rate
No water


E7
E7-S

Order Code **6680542**
Order Code **6680660**

Function Description: Removal of broken instrument

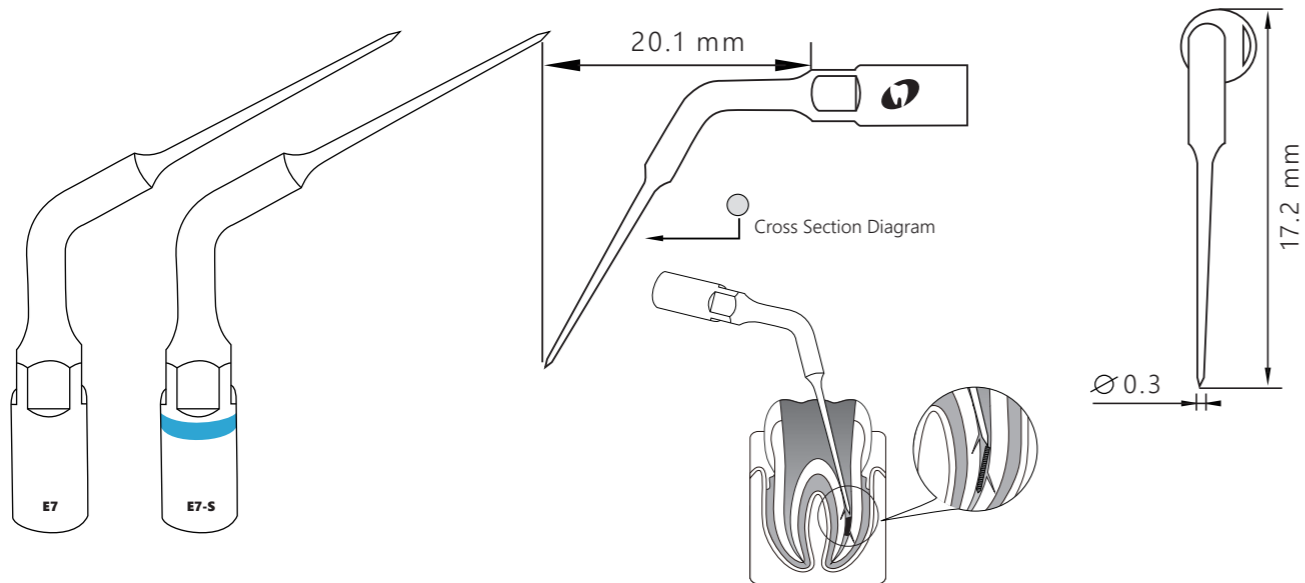

 Mode: E
Power: 1-3


 Recommended
Service Life
50 root canals

 Water Flow Rate
mid&high

E8
E8-S

Order Code **6680608**
Order Code **6680614**

Function Description: Used to brush the canal walls during re-treatment

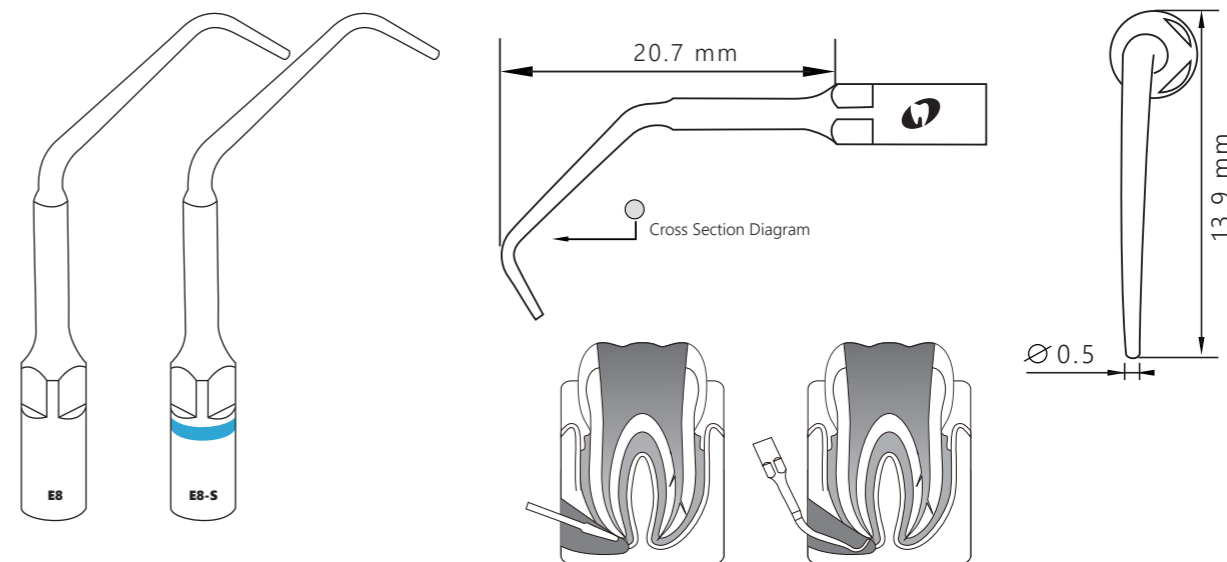

 Mode: E
Power: 1-6


 Recommended
Service Life
50 root canals

 Water Flow Rate
low,mid&high


E9
E9-S


Order Code **6680609**
Order Code **6680615**

Function Description: Flexible, used for irrigation

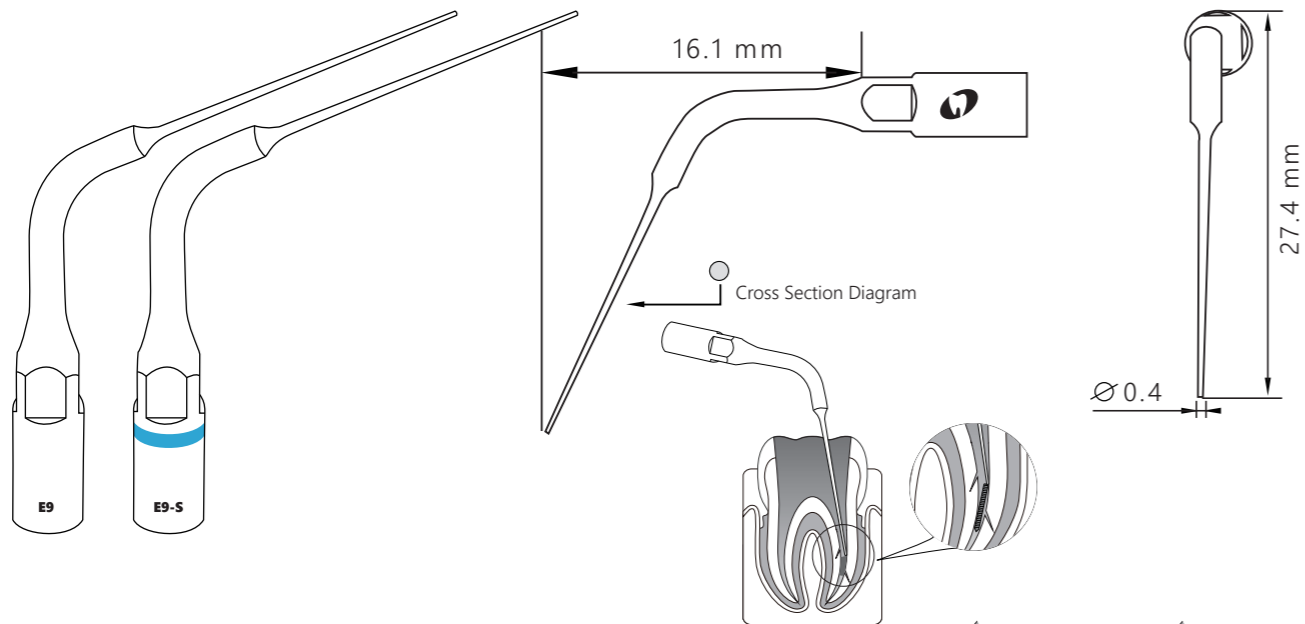

 Mode: E
Power: 1-3


 Recommended
Service Life
20 root canals

 Water Flow Rate
No water

E10
E10-S

Order Code **6680610**
Order Code **6680616**

Function Description: Used for irrigation after preparation

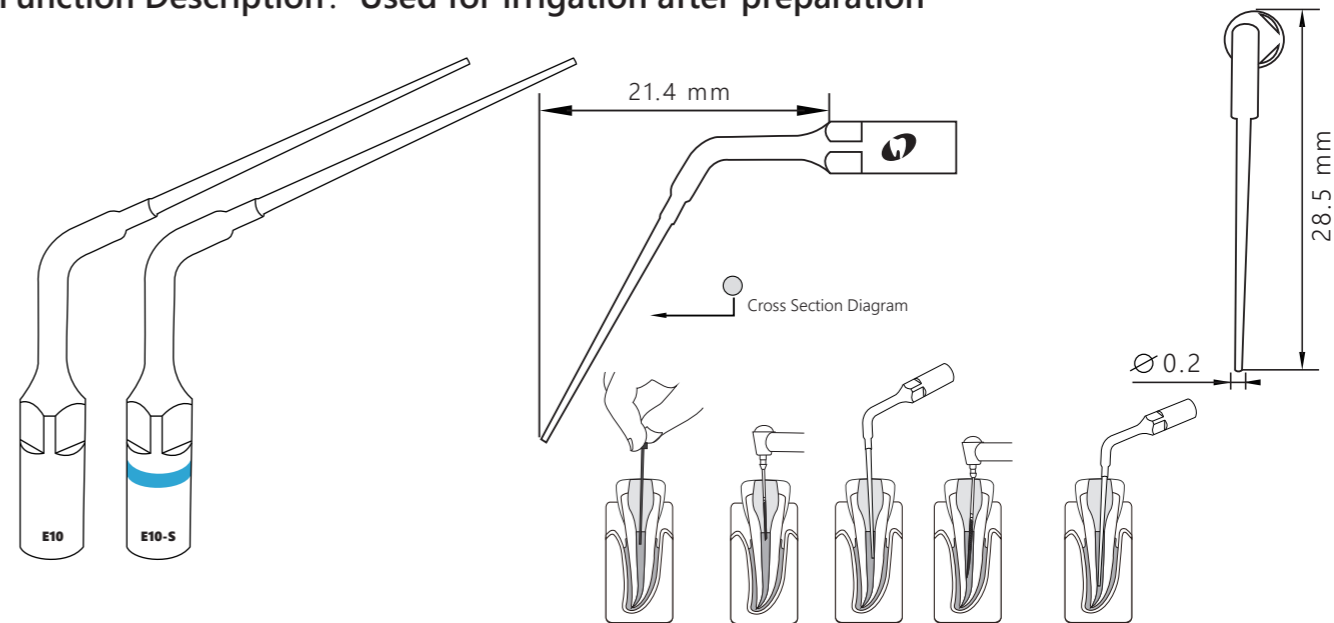

 Mode: E
Power: 1-3


 Recommended
Service Life
50 root canals

 Water Flow Rate
mid&high


E30
E30-S


Order Code **6680611**
Order Code **6680661**

Function Description: Removal of deep foreign particles in canal

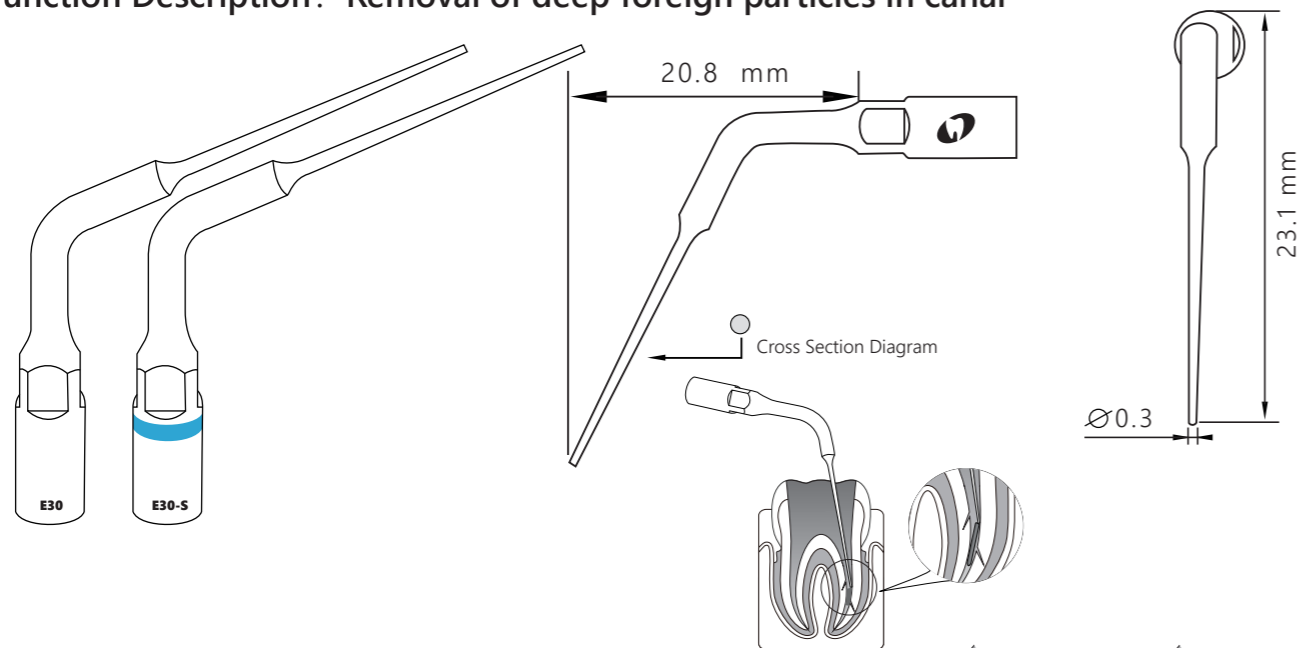

 Mode: E
Power: 1-3


 Recommended
Service Life
50 root canals

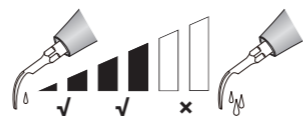 Water Flow Rate
low&mid

E31
E31-S


Order Code **6680612**
Order Code **6680662**

Function Description: Removal of foreign particles in one-third of the canal

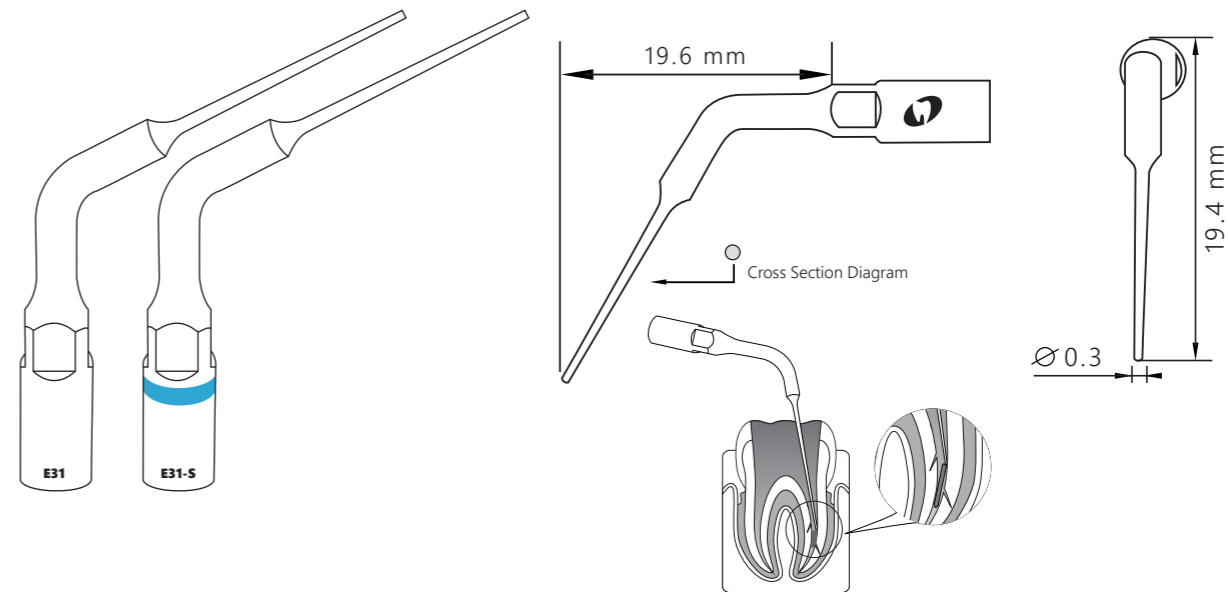

 Mode: E
Power: 1-3


 Recommended
Service Life
50 root canals

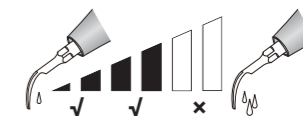 Water Flow Rate
low&mid

E3D
E3-SD

Order Code **6680651**
Order Code **6680638**

Function Description: Enlargement of the canal walls to remove the broken instrument

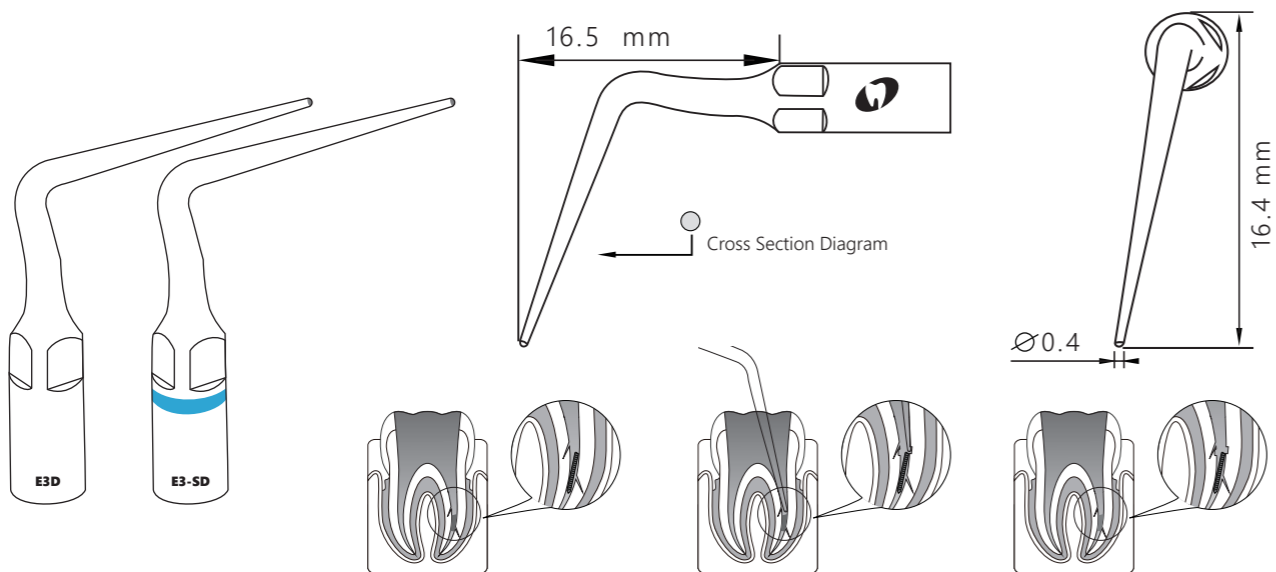

E Mode: E
Power: 1-3


Recommended Service Life
50 root canals

Water Flow Rate
low, mid & high

E4D
E4-SD

Order Code **6680652**
Order Code **6680639**

Function Description: Removal of hard material in canal during re-treatment

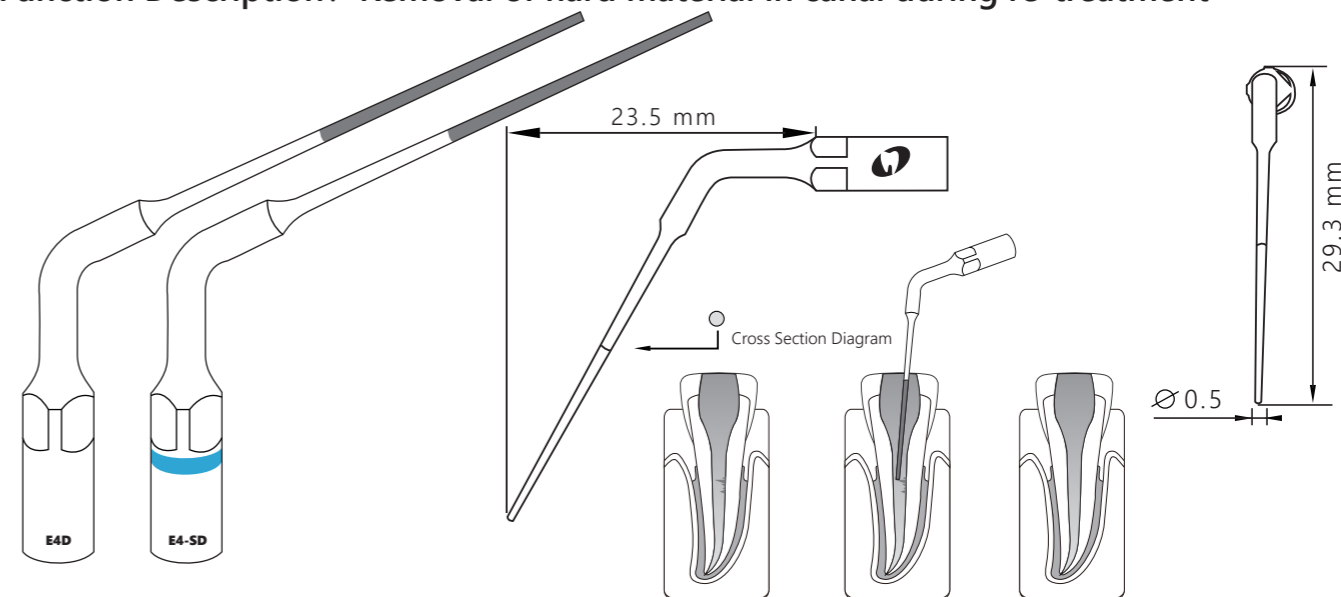

E Mode: E
Power: 1-3


Recommended Service Life
50 root canals

Water Flow Rate
No water


E5D
E5-SD


Order Code **6680653**
Order Code **6680640**

Function Description: Removal of hard material in canal during re-treatment

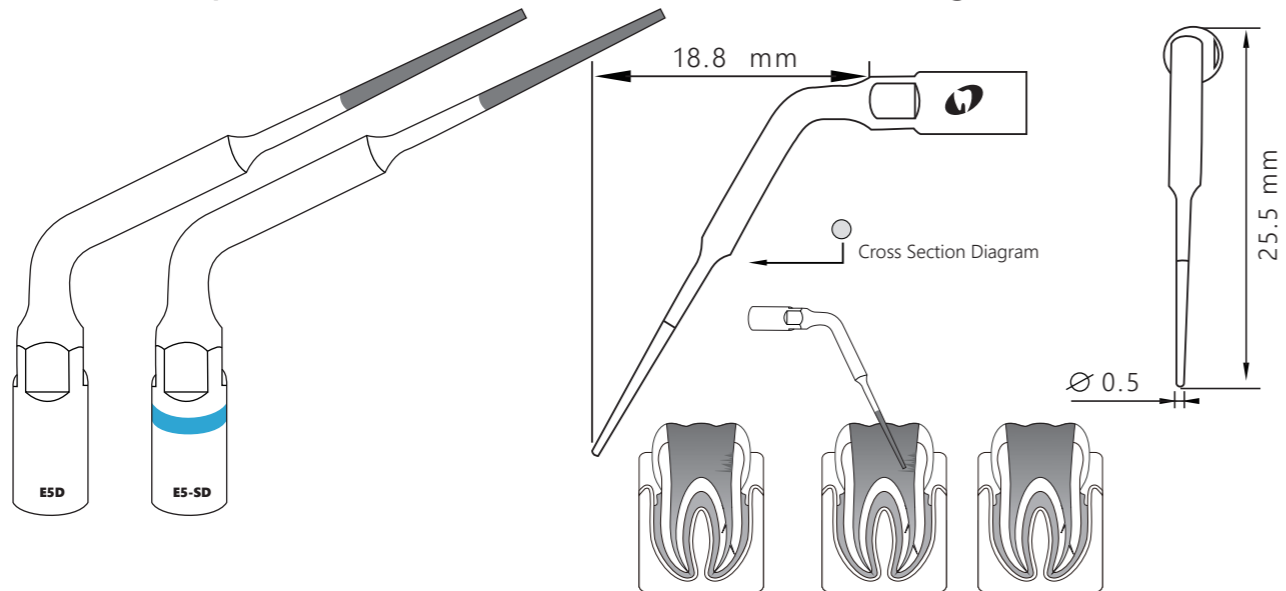

 Mode: E
Power: 1-3


 Recommended
Service Life
50 root canals

 Water Flow Rate
No water

E6D
E6-SD

Order Code **6680632**
Order Code **6680641**

Function Description: Removal of foreign material in cavity to access canal orifice

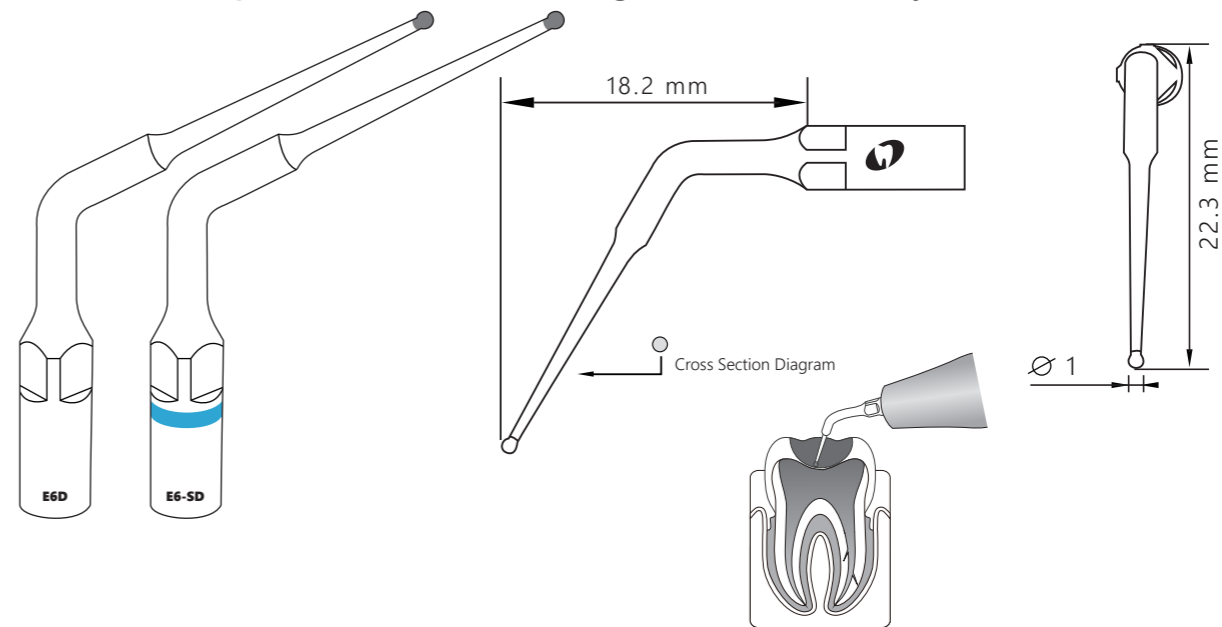

 Mode: E
Power: 1-10


 Recommended
Service Life
50 root canals

 Water Flow Rate
mid&high

E7D
E7-SD

Order Code **6680633**
Order Code **6680642**

Function Description: Removal of calcification and filling material in cavity to access canal orifice during re-treatment

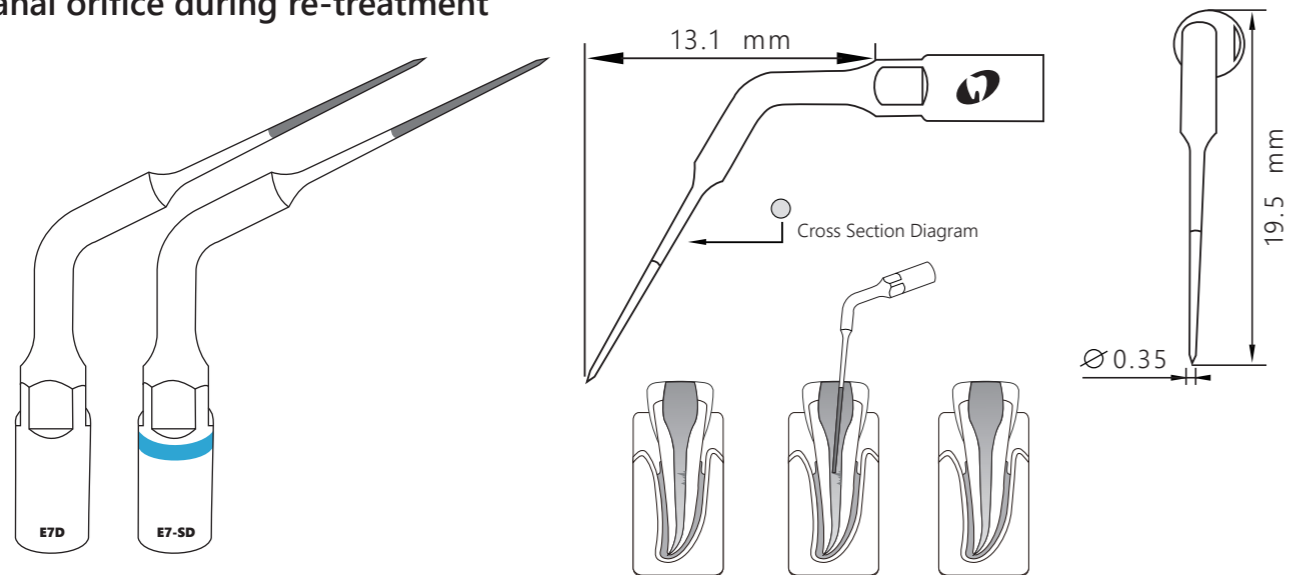

E Mode: E
Power: 1-3


Recommended Service Life
30 root canals

Water Flow Rate
mid&high

E8D
E8-SD

Order Code **6680634**
Order Code **6680643**

Function Description: Used for root canal retrogression, efficient root apical polishing

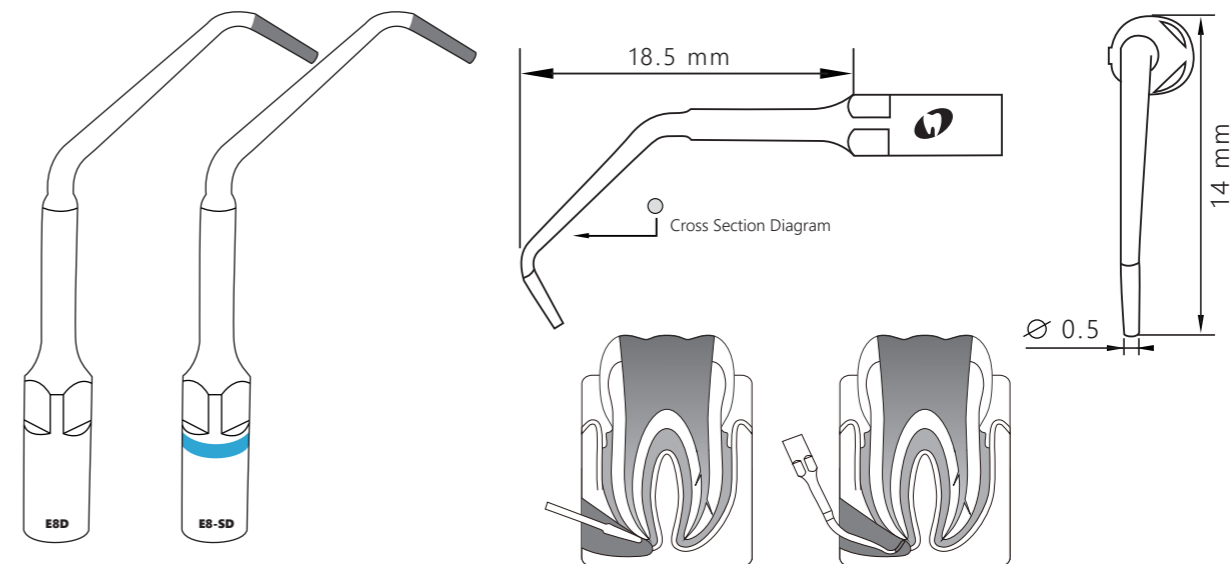

E Mode: E
Power: 1-6

Recommended Service Life
50 root canals




Water Flow Rate
low, mid&high

TC1D
TC1-SD

Order Code **6680648**

Order Code **6680650**


Function Description: Removal cervical caries and caries on the occlusal surface

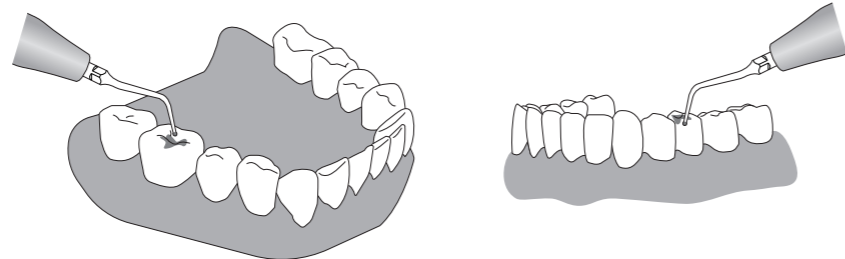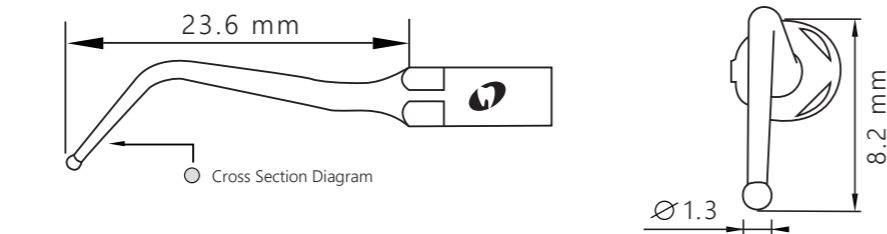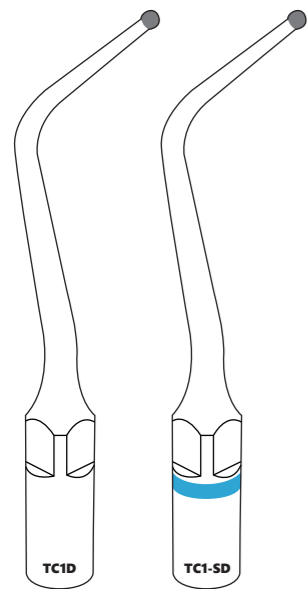

Mode: P
Power: 1-10



Recommended
Service Life
50 Teeth

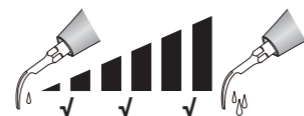

Water Flow Rate
low, mid & high